Product :

THERMAPREP® PLUS

Ref A0176

Ref doc. :

F190314.X / 07 / 1998 – updated 06/2007

CE
0459

		Page
GB	DIRECTION FOR USE	2
F	PROTOCOLE D'UTILISATION	4
E	INSTRUCCIONES DE USO	6
I	ISTRUZIONI PER L'USO	8
D	GEBRAUCHSANWEISUNG	10
DK	BRUGSANVISNING	12
FIN	KÄYTTÖOHJEET	14
S	BRUKSANVISNING	16
NL	GEbruiksvoorschrift	18
P	PROTOCOLO DE UTILIZAÇÃO	20
GR	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ ΓΙΑ	22

THERMAPREP® PLUS

Ref A0176

For dental use only

DIRECTIONS FOR USE

1) Indications for use

The ThermoPrep® Plus Oven has been specially developed for heating ThermoFil® endodontic obturators for use in root canal therapy.

Do not use this oven for any other purpose.

2) Contraindications

There are none known.

3) Warnings

- 1) To prevent fire or electric shock, do not expose this product to moisture,
- 2) Turn off and unplug oven when not in use,
- 3) Do not remove the cover of the oven,
- 4) This oven is designed for use in a dental office. Do not use in any other environment.

4) Precautions

- 1) To reduce the risk of electric shock, do not remove cover, there are no user-serviceable parts inside
- 2) This oven is intended for use in heating gutta-percha coated obturators only. Do not insert anything other than an obturator into the oven.
- 3) The appliance must only be used with the manufacturer's original accessories

5) Adverse reactions

None known

6) Step-by-step instructions

- 1) Plug the oven into a power outlet (110VAC or 220VAC). Then turn the oven on with the power on / off switch located on the back. The green stand-by indicator will light up.
- 2) Put both obturator holders in the upper position by pressing the back of the holder (arrow up), and guiding the holder up with your finger.
- 3) After disinfecting the obturator and setting the silicone stop, place the ThermoFil® endodontic obturator in the left obturator holder. *Regardless of what working length you have determined, the silicone stop must be under the holder.* Make sure the obturator is hanging straight, not at an angle. The obturator must clear all sides of the heating chamber.
- 4) Push the holder down (arrow down) until you hear it click

- 5) Push the button which corresponds to the size of the Thermafil® endodontic obturator you wish to heat. Then push the “start left” button. The obturator is now being heated. The heating time varies – depending on obturator size – from 20 to 45 seconds, and is regulated automatically.

The heating times for each button are as follows:

Size	Signal Time	Tolerance
20 – 25	20 seconds	± 5 seconds
30 – 60	41 seconds	± 7 seconds
70 – 140	45 seconds	± 8 seconds

- 6) After the first signal “beep”, the obturator is ready for use. Push the obturator holder (arrow up) and guide the holder with your finger. Take the obturator carefully out of the holder by pulling it towards, making sure not to scrape the obturator on any part of the holder.
- 7) You may leave the obturator in the ThermaPrep Plus Oven for up to 90 seconds after the first signal “beep”. It will keep the obturator at the appropriate temperature and ready for use. The oven will “beep” every 15 seconds to remind you that the obturator is still in the oven. After 90 seconds, the heating element will switch off automatically.
- 8) As an extra safety precaution, after you’ve taken the obturator out of the holder, switch off the heating element by pushing the start level button again. Keep in mind that you won’t be able to switch the heating element off during the heating cycle.
- 9) If you want to heat more obturators, you may alternate using the left and the right holders to continue your work efficiently. Wait until after the signal “beeps” for the first holder, then you may immediately start to heat the other one. *You will be unable to heat both sides at the same time.*
- 10) When you’re finished using the oven, use the power on/off switch located at the back of the oven to turn it off. Then unplug the oven until you’re ready to use it again.

Cleaning instructions

Before starting any cleaning procedures, disconnect the oven from the power outlet.

Clean oven after each use. Any gutta-percha remaining on the holder or in the heating chamber may be effectively removed with Orange Solvent (generic dental solvent) on a cotton ear bud. The oven’s housing may be cleaned with Orange Solvent.

7) Technical specifications

Voltage rating : 110 VAC or 220 VAC, 50/60 Hertz, range of the supply voltage + / - 10 % , fuse 2 Amps slow.

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